


From Serial Number 5800.18.050

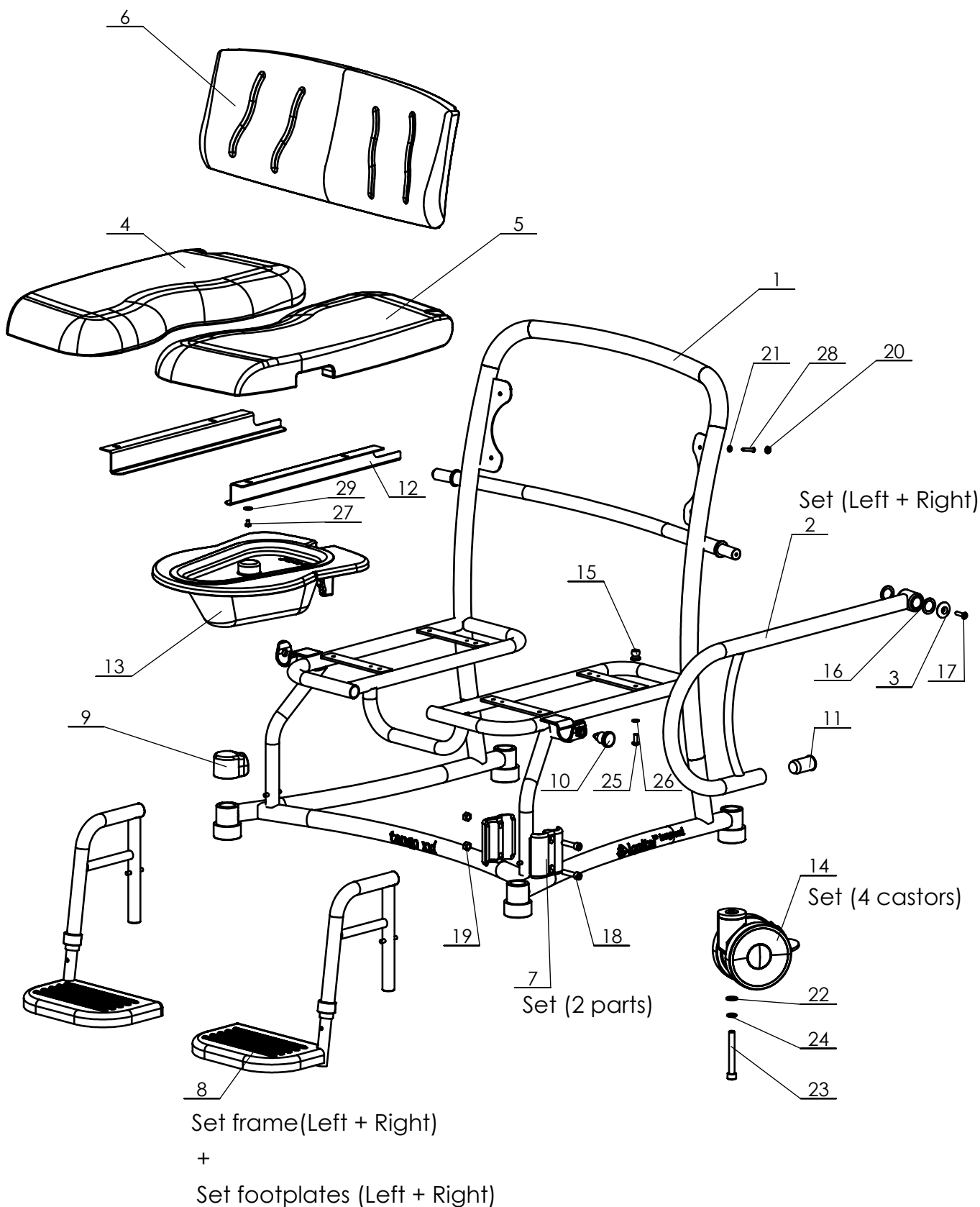

P:\SW-Ftek\Tango XXL 5800\

Signed	ES	Date	07-10-2013
Revised	KA	Date	20-01-2025
Revised	5 /KA	Date	11-11-2022

Overview parts Tango XXL

Copyright reserved
by law

Unit measure-
ment: mm

Lopital Nederland bv

Draw. no. 5800-EV.5

Tango XXL 51005800

Part nr.	QTY.	Art.no.	Part
1	1	70580001	Frame
2	1	70530005	Armrest (L+R)
3	1	Set with no. 1	Rosette
4	1	70530014	Seat XXL (Right)
5	1	70530008	Seat XXL (Left)
6	1	70530009	Backrest
7	4	70519931	Leg support clamp
8	1	70530017 + 70530032	Height adjustable footsupport (Set) + Footplates
9	4	70530006	Protective cover wheel
10	2	70530029	Pull knob
11	2	70530010	tube plug
12	1	70580005	Guide for bedpan set
13	1	69501104	Bedpan + lid
14	3	70530025	Castor double roll set
15	8	70602006	Snaploc
16	4		Washer PA M20
17	2		Countersunk screw M6x25
18	4		Cylinder head screw M8x35
19	4		Locknut M8
20	4		Screw cap M4
21	4		Capring M4
22	4		Washer M10
23	4		Cylinder head screw M10x80 Nord-Lock
24	4		M10
25	8		PAN head cross slotted screw M6x16
26	8		Washer M6
27	4		PAN head cross slotted screw M5x8
28	4		Chipboard screw 4,5x25
29	4		Washer M5 Ø15



P:\SW-Tek\Tango XXL 5800\

Signed	ES	Date	07-10-2013
Revised	KA	Date	20-01-2025
Revised	5 / KA	Date	11-11-2022

Overview parts Tango XXL

Copyright reserved
by law

Unit of measure-
ment: mm

Lopital Nederland bv

Draw. no. 5800-EV.5